

How do I use my Ozempic® pen?

This is a Quick Guide.

Read the Instructions for Use and Important Safety Information before using the pen.

1



Prepare your pen

Check your Ozempic® pen
Read the label to check that your pen contains Ozempic®. Make sure that the Ozempic® medicine in your pen is clear and colorless.

2



Attach a new needle
Tear off the paper tab. Push and turn the needle on until it is tight. Pull off **both** needle caps.

3



Check the Ozempic® flow with each new pen
Turn the dose selector until the dose counter shows the flow check symbol (• • ◀). Press and hold in the dose button until the dose counter shows 0. Make sure a drop appears at the needle tip.

Text PEN to 43967 to view a demonstration.

Please see Important Safety Information throughout, and Prescribing Information and Medication Guide at <https://www.novo-pi.com/ozempic.pdf>

4



Give your injection

Select your dose

Turn the dose selector until the dose counter shows the dose you need to inject (0.25 mg or 0.5 mg).

5



Inject your dose

Insert the needle into your skin. Press and hold down the dose button. **After** the dose counter reaches 0, slowly count to 6. Remove the needle from your skin.

6



After your injection

Carefully remove the needle and place it in a sharps container. Put the pen cap back on the pen.

Important Safety Information

Do not share your Ozempic® pen with other people, even if the needle has been changed. You may give other people a serious infection, or get a serious infection from them.